MANUAL (p. 2)
Blood Pressure Meter

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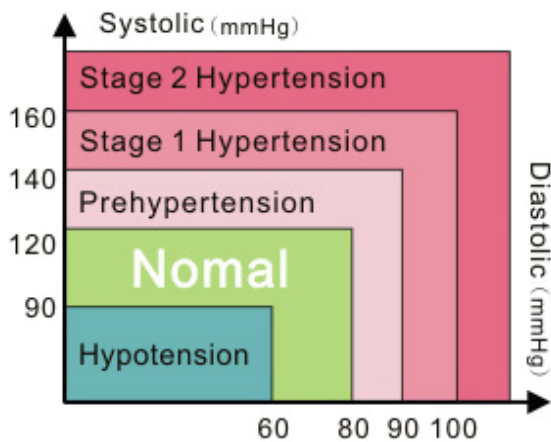
User Instruction Manual

ISH : International Society of Hypertension

Dear Customer,

Thank you for choosing KÖNIG healthcare products. This upper arm type USB blood pressure monitor uses hi-tech intellectual, reliable and accurate measurements for personal healthcare management. No batteries or adaptors are required. Simply connect the USB cable to your computer and you can see your current and previous measurements on your computer screen. This instrument uses the oscillometric method to detect systolic, diastolic, mean value, pulse rate and arrhythmia and displays this information in numeric form and waveform on your computer screen. Please note that this instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

Blood pressure constantly fluctuates, which means that measuring different values is quite normal. These discrepancies can take place at your doctor's, the pharmacy store or in your own home. It is highly recommended to always consult your doctor in case of any unusual value. The image displayed above, "Classification of blood pressure for adults", is a reference data adopted by the International Society of Hypertension (ISH) to evaluate your blood pressure.

The following instructions are divided into 5 sections to ensure that you fully understand every function of this instrument and that you are aware of any important information. Please read it thoroughly and we hope that this instrument will help you on your way to a healthy life.

Index

- 1. Product & Accessories**
 - 1.1 The Main Unit
 - 1.2 Accessories
- 2. USB BPM Software Instructions**
 - 2.1 Installing the Blood Pressure Monitor Software
- 3. Correct Measurement Procedure**
 - 3.1 Using the Arm Cuff Properly
 - 3.2 Correct Posture
 - 3.3 Measurement Procedure
 - 3.4 Storing Your Measurement Results
 - 3.5 Averaging Your Measurement Results
 - 3.6 Blood Pressure Knowledge Guide
 - 3.7 Description of Product Display

4. Product Specifications

5. Notes

5.1 Maintenance & Repair

5.2 Questions and Answers

1. Product & Accessories

1.1 The Main Unit

Cuff Connector, Operation Indicator Light, Power Indicator Light

1.2 Accessories

Arm Cuff, USB Connector, USB Transfer Cable, Storage Bag, USB BPM Software CD

2. USB BPM Software Instructions

Before using the instrument, please insert the attached USB BPM installation CD into your PC. The USB BPM software is compatible with Microsoft Windows 7, Vista, XP or ME. Please read carefully the instructions below to proceed with the USB BPM software installation. The latest USB BPM software version will be released non-periodically on our website.

2.1 Installing the Blood Pressure Monitor Software

Execute the USB BPM Software CD to start the installation (as in the picture shown above)

Click on "Next" and the following picture will be shown on the screen.

Click on "Next" until the following picture appears on the screen.

When the software has been installed successfully, click on "Finish".

The USB BPM icon is placed permanently on the bottom right corner of the toolbar after the installation.

3. Correct measurement procedure

Before you start using this instrument, please take a look at the suggestions below to avoid discrepancies in your measurements.

- Always measure on your left arm or left wrist
- Avoid exercising, eating, drinking or smoking
- 5-minute intervals between measurements is recommended

3.1 Using the Arm Cuff Properly

Hold your hand open with the palm up and place the arm cuff around your upper arm.

Maintain a distance of about 2-3 cm above the elbow.

Place the arm cuff directly against the skin to avoid measurement errors.

Place the arm cuff over your upper arm and then fasten the arm cuff firmly with the Velcro tape.

3.2 Correct Posture

Sit comfortably and rest your arm on the table. Place the centre of the arm cuff at the same height as your heart.

3.3 Measurement Procedure

It is recommended to close all other programmes while running the BPM programme. Connect the USB BPM to your computer or laptop using the supplied USB transfer cable. The Link Indicator Light will light up for as long as the programme is running. After the computer has detected the BPM program, it will start automatically. The Link Indicator Light on the right top corner of the screen will light up while the BPM connects and interacts with the computer. You can now start measuring your blood pressure. Press "Start" to start the measuring process. The Link Indicator Light will flash until this process is completed. The results will be displayed on screen in all kinds of figures, including the pressure waveforms.

3.4 Storing your Measurement Results

To store your measurement results, you need to enter a filename in the blank space on the right hand side of the screen and press "Save File". The measurement results will be saved on your computer and is displayed in graphics.

3.5 Averaging your Measurement Results

To know your average measurement results, select the measurement records you want to calculate and click the AVG button. It will start to calculate the mean value automatically.

3.6 Blood Pressure Knowledge Guide

This instruction guide provides information about blood pressure and related health knowledge. Users are able to study this information and use it to learn more about their health.

3.7 Description of Product Display

1. Pressure Waveform
 2. Integrated Pressure Waveform
 3. Average Measurement Results
 4. Blood Pressure Knowledge
 5. Systolic Pressure
 6. Diastolic Pressure
 7. Mean Arterial Blood Pressure.
(Diastolic Blood Pressure + $1/3$ (Systolic Blood Pressure – Diastolic Blood Pressure))
 8. Pulse Pressure. (20 mmHg < Normal Pulse Pressure < 60 mmHg)
 9. Pulse Rate
 10. Start Button: Start the Measurement
 11. Stop Button: Compulsively Stop the Measurement
 12. Exit Button: Leave the Measurement Program
 13. Save File Button: Save measurement results to PC in jpeg file format
 14. Filename Column: Key in a file name
 15. Link Indicator Light
-

4. Product Specifications

Name	Arm Type USB Blood Pressure Monitor
Model No.	HC-BLDPRESS250
Product Dimensions	Main Unit: 158 (L) × 50 (W) × 46 (H) mm
	Arm Cuff: 470 (L) × 140 (W) mm
Measurable Circumference of Wrist	Approx. 22 ~ 32 cm
Unit Weight	Approx. 150g
Measurement Range	Pressure: 0~300 mmHg Measuring Interval: 1 mmHg
	Pulse Rate: 30~200/min.
Accuracy	Pressure: within ±4 mmHg of reading
	Pulse Rate: within ±5% of reading
Measurement	Oscillometric Method

Power Supply	DC 5V from USB port of your computer
Power Consumption	2W
Operating System	Microsoft Windows 7, Vista, XP or 2000.
Operating Conditions	Temperature: 10 ~ 40°C Humidity: 30%~85% RH (without condensation)
Storage Conditions	Temperature: -20 ~ 50°C Humidity: 10%~95% RH (without condensation)
Accessories	USB Transfer Cable / Storage Bag / Arm Cuff / User Instruction Manual / USB BPM Software CD

5. Notes

5.1 Maintenance and Repair

This instrument includes sensitive components. Please make sure it is always protected from moisture, impact, dropping, dust and direct sunlight, and handle it with care to ensure prolonged usage. It is recommended to clean the surface of the instrument and the arm cuff with a soft and dry cloth.

5.2 Questions and Answers

- Q1: The computer did not run the measurement programme automatically after the unit had been connected.
A1: Please click on the icon on the Toolbar at the bottom right corner of the screen to execute the program manually.
- Q2: What is the standard specification of the attached USB Transfer Cable?
A2: An additional USB 2.0 extension cable is compatible with the monitor.
The total length of the cable must not exceed 120 cm.
- Q3: The cuff keeps pumping up and is not stopping.
A3: Press "Stop" to stop the measurement process.
Close the software and restart it.
Close all other programmes and re-execute the measurement process.
- Q4: The software cannot be displayed normally on screen.
A4: Check if you have installed the software correctly, and if your computer system supports the software.

Safety precautions:

To reduce risk of electric shock, this product should **ONLY** be opened by an authorized technician when service is required. Disconnect the product from mains and other equipment if a problem should occur. Do not expose the product to water or moisture.

Maintenance:

Clean only with a dry cloth. Do not use cleaning solvents or abrasives.

Warranty:

No guarantee or liability can be accepted for any changes and modifications of the product or damage caused due to incorrect use of this product.

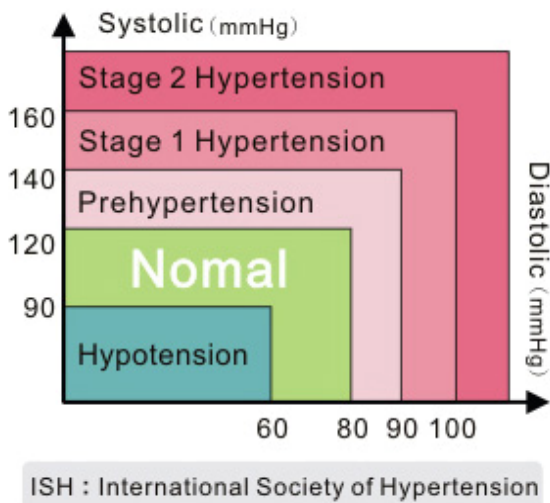
General:

- Designs and specifications are subject to change without notice.
- All logos brands and product names are trademarks or registered trademarks of their respective holders and are hereby recognized as such.
- This manual was produced with care. However, no rights can be derived. König Electronic can not accept liability for any errors in this manual or their consequences.
- Keep this manual and packaging for future reference.

Attention:

This product is marked with this symbol. It means that used electrical and electronic products should not be mixed with general household waste. There is a separate collections system for these products.

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Copyright ©
NEDIS B.V.